


# BD Sharps Collector with Sliding Top

Distributed by  
**ThermaSolutions**


Product # ANC-3501

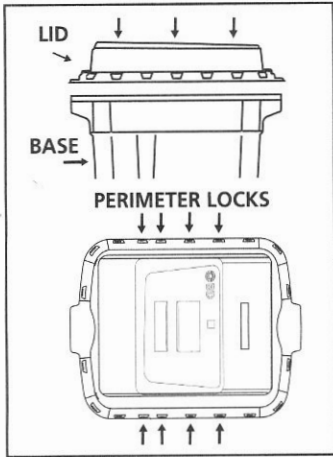

## Assembly

Position the Lid over the base and press down firmly on two corners. Audible clicks can be heard as locks engage.

Slide hands around the perimeter of the Lid until hands are centered over the perimeter locks and squeeze the Lid and Base between the palm and finger tips to engage the locks. Repeat this process until all perimeter locks are fully engaged.

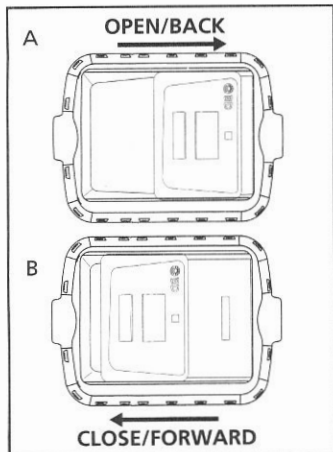

## Use

A. Slide door back to open for disposal (Universal Precautions recommend wearing gloves). If using the Foot Operated Trolley, depress foot pedal to open.

B. Slide door forward to cover opening and engage temporary closure. Be careful not to engage permanent locks. If using Foot Operated Trolley, release foot pedal to close door and engage temporary closure.




## Final Closure

Grasp front edge of sliding door with finger tips, place thumbs on the outside of top, squeeze fingers and thumbs together to engage final closure. An audible click will occur.

## Disposal

Be sure Lid is closed and secure before transport. Always use handles for transport. Dispose of collector in accordance with local regulations.

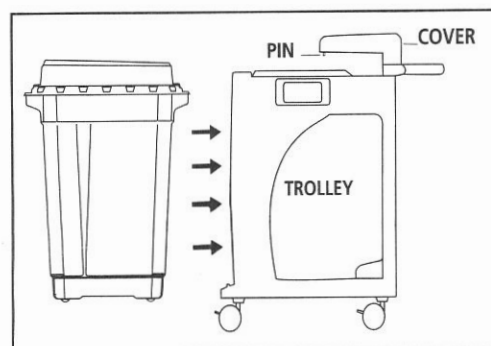

### For use with BD Trolley:

**Installation:** Slide assembled sharps collector into open end of trolley.

**Removal:** Permanently close and lock the sharps collector before removal from the Trolley. Lift handle of sharps collector, at open end of trolley, slide closed collector out.

### For use with BD Foot Operated Trolley:

**Installation:** Pivot Trolley cover out of the way and slide assembled sharps collector into open end of trolley with access opening of collector at open end of trolley. Make sure that the sliding door is in the temporary closed position. Lower the Trolley cover and engage the pin into the receptacle on the sliding door.

**Removal:** Raise the Trolley cover to disengage pin from door. Permanently close and lock the sharps collector before removal. Lift handle of sharps collector at open end of trolley and slide closed collector out of trolley.

## Safety Tips

- Be sure collector is fully assembled and not damaged before use.
- Do not reach into the collector while in use.
- Contents are visible through the translucent Lid.
- Ensure that the collector opening is clear of sharps waste before depositing more waste or engaging the final closure.

- Discontinue use of the collector when contents reach the "Fill Line".
- Do not overfill or force sharps waste into collector. **Collector is puncture resistant, not puncture proof.**
- Collector is designed for single use only.

QUESTIONS? COMMENTS?  
CALL 1-888-237-2762

Do Not  
Recap



BD Medical  
1 Becton Drive  
Franklin Lakes, NJ  
07417-1884

For More Information Refer to **NIOSH** Publication "Selecting, Evaluating and Using Sharps Disposal Containers" [www.cdc.gov/niosh](http://www.cdc.gov/niosh)  
BD, BD Logo, and all other trademarks are property of Becton, Dickinson and Company. © 2010 BD

IN1496/5

Distributed by

**ThermaSolutions**

**Chemo Waste Bin (ANC-3501) Reorder Information: [orders@thermasolutions.com](mailto:orders@thermasolutions.com)**

IFU-3501.revA